

Waterlily hygiene mattress

The environmentally friendly mattress

The Waterlily hygiene mattress is made with materials which reduce the influence on the environment through the entire process, from raw materials to recycling, without sacrificing its hygienic qualities, fire safety or durability.

The Waterlily hygiene mattress reduces the possible pollution of the environment up to 75% as compared to common mattresses used in health care.

Waterlily has received the EPA (Environment Protection Agency) "Stratospheric Protection Award".

The Waterlily hygiene mattress has extra high-density foam which guarantees comfort and performance throughout the entire life span of the mattress.

The Waterlily hygiene mattress has proven to be very effective for preventing bedsores, caused by pressure, through its unique pressure-reducing qualities. This makes it suitable for use by all kinds of patients/users.



UNDVIK
FAKA FÖREHÅLL



INGA
ANDRA PLASTPRODUKTER
I DIREKT KONTAKT MED
LÄNTERESTYMMET.



UNDVIK
AUTOMATISK
AVTVÄNING.



ALDRIG
KLORINATICA
RENGÖRINGSMEDEL



VID SKADA
ANVÄND VÅR SPECIAL
TEGP LENTEX.



VID ODÖR
RENGÖR MED CITRUS-
ESSENS PÅ FUKTIG TRASA.



DESINFICERA
MED ALKOHOLBASERAT
RENGÖRINGSMEDEL
MED TENDLER.



RENGÖRING
ANVÄND TÄLLBÅSERAT
RENGÖRINGSMEDEL
OBS! EJ KONCENTRAT!

CE • FREONFRI • UPPFYLLED BRANDKRÄV NT FIRE 037; A+B

WATERLILY
with **Lentex**


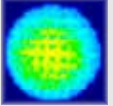

Bromsvägen 3, 591 60 Örnsköldsvik
Tel. +46 (0)660 22 28 50, www.jarven.se



Waterlily hygiene mattress

The environmental choice

Mattress	Large Indenter 300 N	Large Indenter 600 N	Small Indenter 40 N	The method of measuring
Soft Euroflex	28.9	61.1	52.1	To avoid the individual impact of different people when measuring pressure, Swerea IVF has developed pressure indenters, simulating the shape of selected parts of the human body. An objective evaluation method has been created by applying pressure equal to a medium-sized human body on the indenters and measuring with pressure mats. The method of measurement is described in a proposal to the Swedish standard with the designation SS 876 00 13
Waterlily	30.1	55.9	56.4	
Solid	30.6	46.8	53.7	
Exclusive	35.9	53.0	59.2	
Soft Comfort	37.5	53.6	60.5	
Comfort	41.9	54.5	64.5	
<ul style="list-style-type: none"> All measurements in mm Hg. Low number equal low pressure. The number on "Large Indenter 600 N" is of special importance for heavier users. 				

Please contact Järven Health Care or SWEREA/IVF for more information about the test method

Features of the Waterlily hygiene mattress

The Waterlily hygiene mattress uses extra high density foam (54 kg/m³) which guarantees a preserved level of comfort for many years.

The Waterlily hygiene mattress is laminated with our unique surface layer, Lentex, which repels moisture and fluid. Lentex is a very flexible hygienic surface layer which contributes to the Waterlily hygiene mattress's excellent pressure reducing qualities. The white colour aids early discovery of soiling on the surface of the mattress, an extra help for achieving better hygiene.

Waterlily hygiene mattress protects against wounds caused by pressure through its unique pressreducing qualities and that makes it suitable for all kind of patients/users.

Articlenumber	Standard sizes	Article number	Standard sizes
5050	190-205x80x10 cm	5000	190-205x80x12 cm
5051	190-205x85x10 cm	5001	190-205x85x12 cm
5052	190-205x90x10 cm	5002	190-205x90x12 cm

Alternate sizes are available on special order.
CE-certified. Freon free.

Warranty: 10 cm - 1 year for structural fatigue (Swedish standard B value test according to SS 876 00 11).
12 cm - 4 year for structural fatigue (Swedish standard B value test according to SS 876 00 11).
5 years for the Lentex surface adhesion.

All Järven mattresses are marked with the manufacturing year and week as well as the order number. This can be found on the short side of the mattress.

PRISMA - Your health care mattress guide

By using our *PRISMA - mattress guide* you will be able to choose the mattress that is right for you and your needs. For further information, please contact us.

PRISMA Your health care mattress guide.

COMFORT



HYGIENE



DURABILITY



PATIENT/USER WEIGHT



ENVIRONMENT



FIRE SAFETY REQ



Järven Health Care

Bromsvägen 3 • SE-891 60 Örnsköldsvik, SWEDEN
Telephone: +46(0)660-22 28 50 • **Fax:** +46(0)660-824 10
Internet: www.jarven.se • **E-mail:** healthcare@jarven.se